— ABSORBABLE THREADED SUTURE
ANCHOR

MEMOSORB™

SURGICAL TECHNIQUE

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Depuis 1714


MEMOSORB™ SYSTEM

- The MEMOSORB™ system was designed to reattach soft tissues (tendon or ligament) to bone.
- Each anchor is supplied on an implant holder and with a single-use tap that will be used to prepare the fixation point in bone.


PLANNING

- Plan the position of the anchor near the soft tissues being reattached.
- For an effective hold, the anchor must cross one bone cortex (monocortical hold).


TAPPING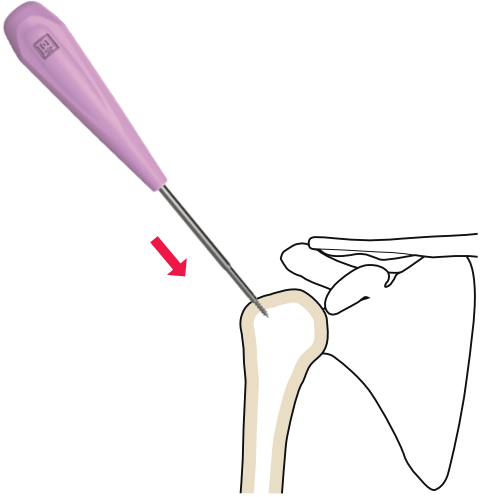



- Prepare the hole using the single-use tap (pink handle) by inserting it to the depth reference mark.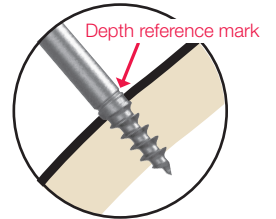


PLACEMENT OF DEFINITIVE IMPLANT

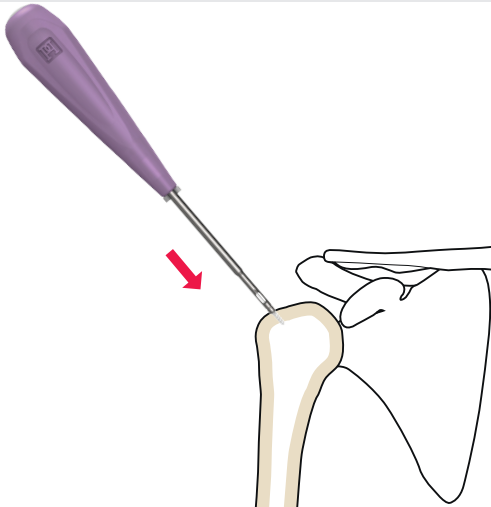
- Using the implant holder (purple handle), screw the anchor until the depth mark is reached.
- Release the sutures and needles from the implant holder.
- Check the anchor's hold in the bone. Make sure the suture slides in the eyelet and that it does not get cut on a sharp bone edge.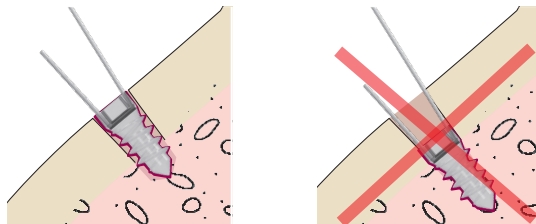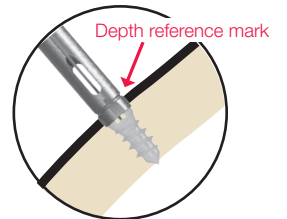

SUTURE

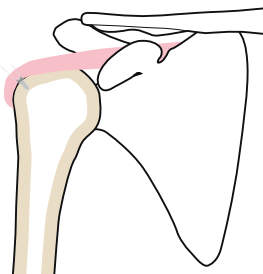- Reattach the soft tissues using one (or more) of the suture/needle combinations, then knot them to bring the soft tissues as close as possible to the anchor. Finish by tightening the suture knots.
- Depending on the size of the tissue fragment being repaired, several anchors may be needed to ensure effective reattachment.

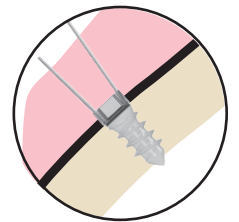lépine Algérie
9 rue philosophe Tabrizi
Les sources
Bir Mourad Rais - ALGER
lepine-algerie@groupe-lepine.com

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